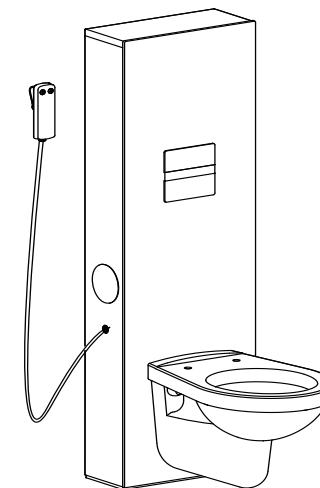
**WISA®**

# XS VARIOTRONIC

Handleiding	<b>NL</b>
Bedienungsanleitung	<b>DE</b>
Manual	<b>EN</b>
Manuel d'utilisation	<b>FR</b>

ART.NR. 1449979170 VERSIE 2019.04.17

**WISA®** World of Water

**FLUIDMASTER BV**

Driepoortenweg 5, 6827 BP Arnhem, Postbus 2194, 6802 CD Arnhem, The Netherlands  
T +31 (0)26 362 90 20 F +31 (0)26 361 73 75 [infolnl@fluidmaster.com](mailto:infolnl@fluidmaster.com) [www.wisa-sanitair.com](http://www.wisa-sanitair.com)

**FR**

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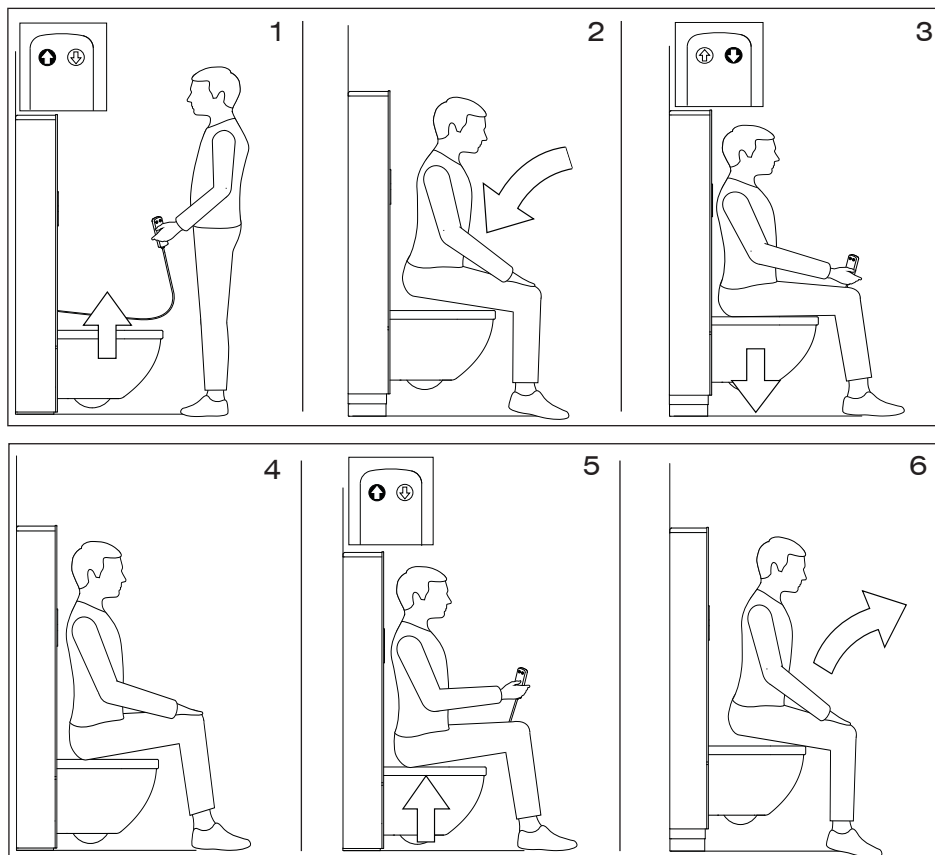
### 6) Technical specifications

XS flushing cistern	6/3 litres
Height adjustment range	200 mm
Mains current connection	AC 100-240V~ / 50-60 Hz
Electricity supply	Max. 3.15 A
Motor connection voltage	24 V
Complete degree of protection	IP44
Motor degree of protection	IP66
Max. user weight	175 kg
Movement speed (no load)	12 mm/sec
Max. weight on support arm (optional)	100 kg

EN

#### Disclaimer

The Wisa B.V. company accepts no liability for personal injury or property damage due to incorrect installation, use or maintenance, in particular, caused by actions contrary to the warnings and assembly instructions.

#### 4) Trouble shooting

EN

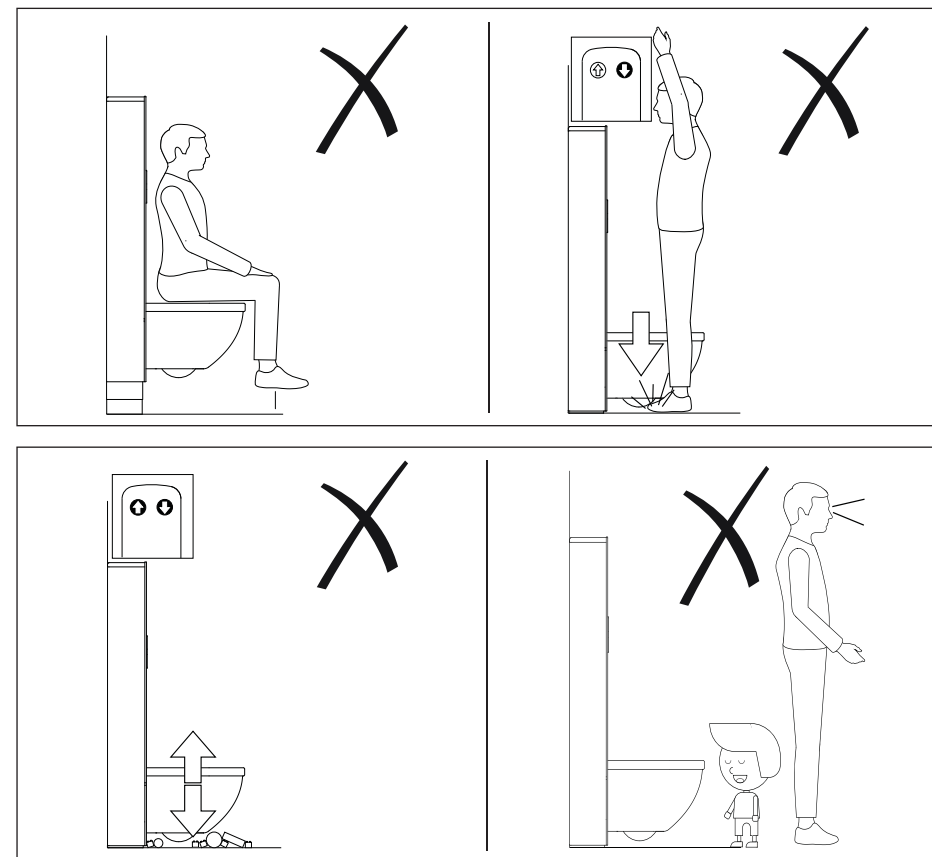
Error	Possible cause	Solution
No vertical movement	Controls not connected properly	Check the remote control's plug connection
	Power supply interrupted	Check fuse in fuse box
		Check plug connection
	Break in control element cord	Have the control element replaced by an authorised fitter
	Push-button control element broken	Have the control element replaced by an authorised fitter
	Bearing jams	Check bearing and, if necessary, have it replaced by an authorised fitter
	Control unit broken	Have the unit replaced by an authorised fitter
Motor broken	Have the unit replaced by an authorised fitter	
Toilet does not flush	Water supply interrupted	Open the stop valve in the reservoir
	Kink in water supply hose	Fit the hose according to the instructions and, if necessary, replace it
Toilet keeps running	Dirty or defective intake-valve membrane	Check membrane and, if necessary, have it replaced by an authorised fitter
	Dirty/defective discharge mechanism	Check bottom valve and, if necessary, have it replaced
Toilet flush is not sufficient	Water level in the reservoir is too low	Adjust the water level in the tank to align with the 6 liter water line
	Increase the amount of flush water	Slide the change-over knob downwards on the discharge mechanism

#### Optional

Error	Possible cause	Solution
Support bar falls into the horizontal position	Insufficient clamping between support bar and element	Tighten the bolts on the support bar
Support bar remains in vertical position	Too much clamping between support bar and element	Loosen the bolts on the support bar

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
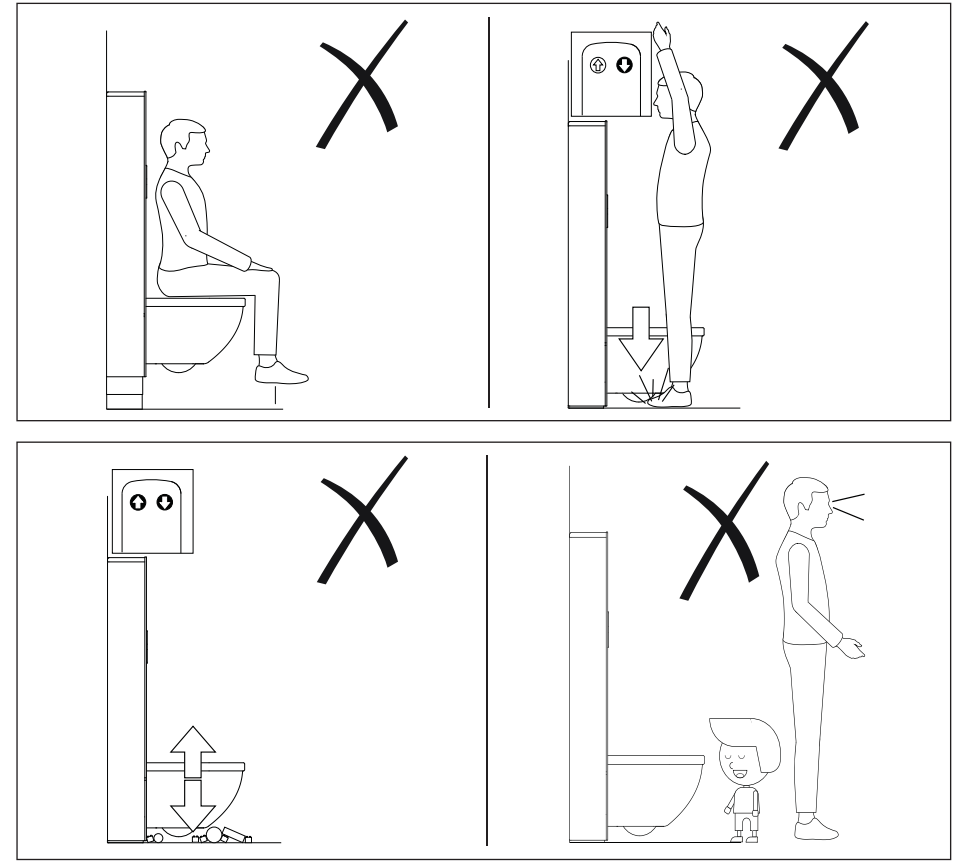
### B Incorrect use

## 1) Operation

Figure series A shows how to operate/use the toilet to achieve the optimum position. Optimum sitting height will depend on the user's circumstances. If in doubt, consult your therapist.

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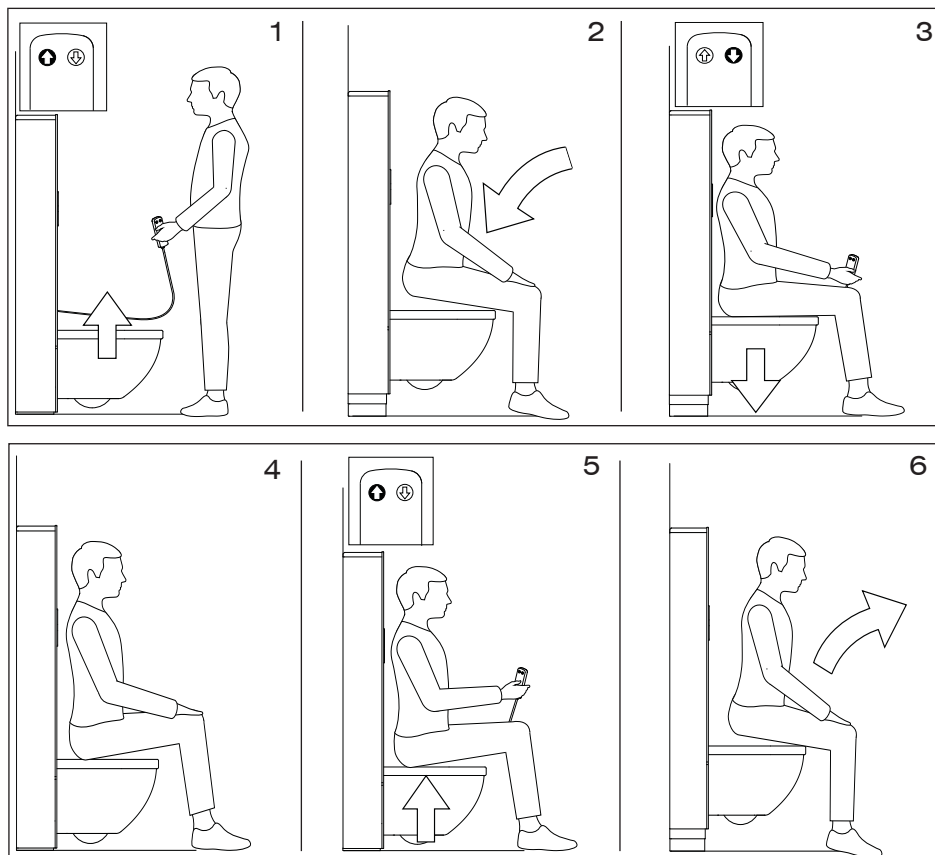**Intended use**

The Wisa high-low Variotronic toilet is designed for the users with difficulties getting on and off a standard toilet.

If you move out, please pass this manual on to the next occupant.

**Safety instructions**

Repairs and maintenance must be carried out by a certified fitter.

Only use original parts for repairs. Do not use any lubricant. Making alterations to the product or software is prohibited.

**Disposal methods**

This section relates to the disposal of used electrical and electronic equipment and batteries (in effect in countries with a separate collection system for such equipment).

The symbol (crossed-out dustbin) on the product or the packaging indicates that the product must not be treated as domestic waste, but taken to a collection station for recycling electrical and electronic equipment and batteries.

**Cleaning instructions**

Only use a soft cloth and a neutral, non-abrasive liquid cleanser to clean the product.

**Note!**

Do not use cloths with synthetic fibres, scourers, scourers with metal wires, solid or liquid detergent with abrasives, alcohol, (household) acids, acetone, or other solvents as these may damage the surface.

1. Clean with a moist (not wet) soft cloth.
2. Use a soft, dry cloth to wipe dry.